




# TOUCH OPERATION

## for Malmet WDS Models

(for Hands Free operation, please refer to WDS Operation, Maintenance and Installation manual)

### STEP 1 OPEN DOOR

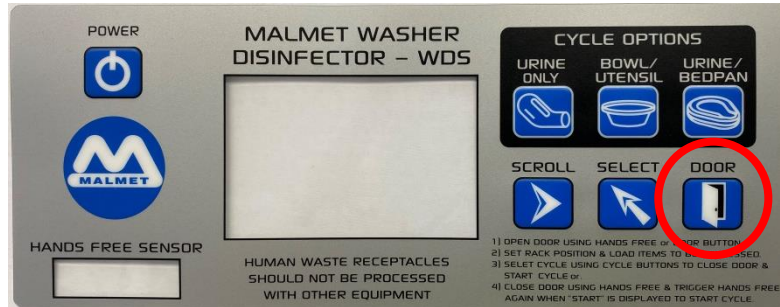

Press "DOOR" Button

Note: Display flashes between "COOLING" and "WAITING". If the chamber temperature is too hot; the door cannot be opened until chamber is at a safe temperature

### STEP 2 SET RACK AS REQUIRED & LOAD ITEMS




Urine/Bedpan rack in position with items loaded



Bowl rack in position with items loaded

Only **ONE** of the above racks can be in position at any one time

### STEP 3 START CYCLE

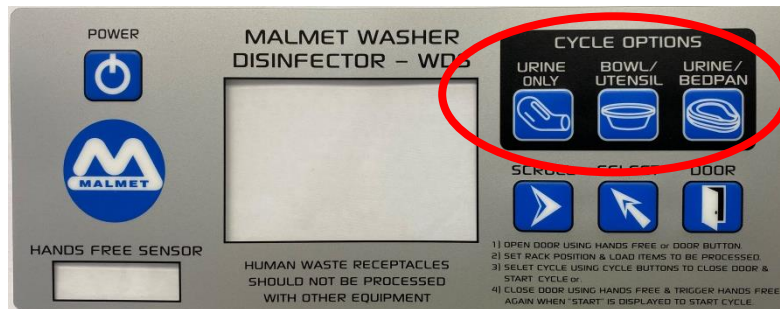

Press required "CYCLE OPTIONS" Button

**Bowl/Utensil** – If bowl rack is in position and loaded  
**Urinal/Bedpan** – If bedpan rack is in position and loaded

### STEP 4 COMPLETED CYCLE



After every successful cycle "COMPLETED" will be displayed until the door is opened.

**WARNING:** If this is not displayed, a cycle has NOT been run