



# WDS INSTRUCTIONS



**IN AN EMERGENCY,  
PRESS THE EMERGENCY CYCLE STOP BUTTON**


**ONCE THE CYCLE HAS STARTED, IT CANNOT BE INTERRUPTED**

**BE AWARE THAT THIS WILL CAUSE THE DEVICE TO GO TO FAULT AND A  
SERVICE TECHNICIAN WILL BE REQUIRED TO RESET DEVICE**



**DO NOT PRESS POWER OFF DURING CYCLES**


TO SELECT RELEVANT CYCLE

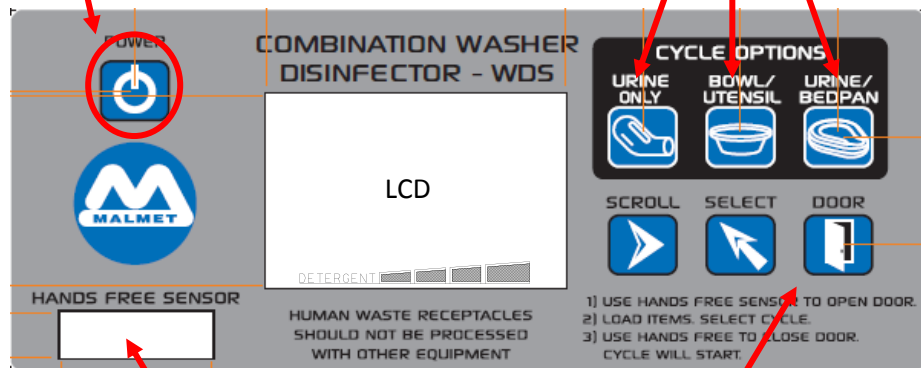

HAND FREE SENSOR  
TO OPEN OR CLOSE DOOR

MANUALLY OPEN & CLOSE DOOR

## STEP 1 OPEN DOOR

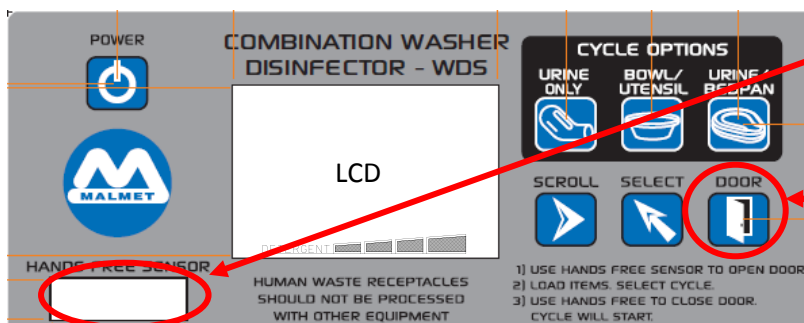

**HOLD HAND OVER  
'HANDS FREE SENSOR'  
Or  
'DOOR' BUTTON**



# WDS INSTRUCTIONS

## FOR URINAL BOTTLES ONLY



### STEP 2 SET RACKS AS REQUIRED



BEDPAN RACK CAN BE UP OR DOWN



BOWL RACK MUST BE UP AGAINST BACK WALL OF CHAMBER

### STEP 3 LOAD URINAL BOTTLES




### STEP 4 SELECT CYCLE



### STEP 5 CLOSE DOOR



HANDS FREE SENSOR

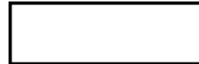

or

## CYCLE STARTS AUTOMATICALLY



# WDS INSTRUCTIONS

## FOR BOWLS AND/OR UTENSILS



### STEP 2 SET RACKS AS REQUIRED



**BEDPAN RACK MUST BE IN THE DOWN POSITION**



**BOWL RACK MUST BE IN DOWN POSITION**

### STEP 3 LOAD BOWLS AND UTENSILS



**KIDNEY DISHES ON KIDNEY DISH RACK**



**LARGE BOWLS**



**MEDIUM BOWLS**

### STEP 4 SELECT CYCLE




### STEP 5 CLOSE DOOR



HANDS FREE SENSOR

or

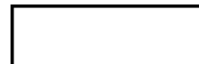

## CYCLE STARTS AUTOMATICALLY



# WDS INSTRUCTIONS

## FOR URINAL BOTTLES AND/OR BEDPANS



### STEP 2 SET RACKS AS REQUIRED



BEDPAN RACK UP & LOCK IN POSITION




BOWL RACK MUST BE UP AGAINST BACK WALL OF CHAMBER

### STEP 3 LOAD BEDPANS AND/OR URINAL BOTTLES



BEDPAN LIDS INTO BACK OF BOWL RACK

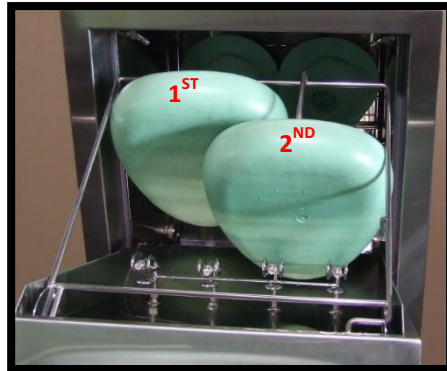

BEDPAN OR SLIPPER PANS ONTO BOTTOM HOLDER OF RACK



URINAL BOTTLES IN RACK

### STEP 4 SELECT CYCLE




### STEP 5 CLOSE DOOR



HANDS FREE SENSOR

OR

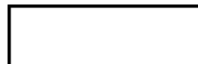

## CYCLE STARTS AUTOMATICALLY